


# Coronavirus (COVID-19) Rapid Test

For Medical Professional Use Only

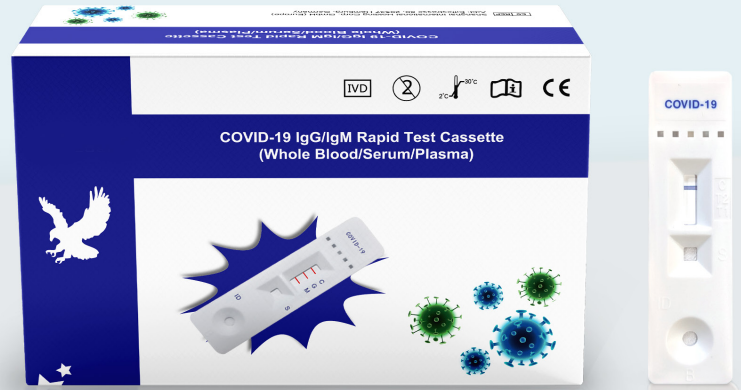

**NOW AVAILABLE FOR PRE-ORDER\***

Our easy-to-use COVID-19 test kit provides results in just **10 minutes**



COST-EFFECTIVE



FAST



ACCURATE

## HOW ACCURATE IS THE CORONAVIRUS (COVID-19) RAPID TEST?

Our test has shown in clinical evaluation to have a total agreement of **97.19%** and a kappa value of 0.94.

TEST REAGENT	REFERENCE REAGENT		TOTAL
	POSITIVE	NEGATIVE	
Positive	245	4	249
Negative	16	439	455
Total	261	443	704

Results of the first clinical evaluation, published on March 12, 2020

Clinical evaluation took place across 10 hospitals in China during February 2020 and included 704 tests. Ongoing clinical evaluation and other information is available upon request.

## COVID-19 "CORONAVIRUS" IGG/IGM RAPID TEST KIT

To support public health & safety, we are offering a proven, cost-effective option to detect and help stop the spread of coronavirus.

- Rapid results in 10 minutes
- Following the incubation period, IgM may appear in blood within 3-5 days. IgG will appear as soon as 1-2 weeks.
- Sample: Test can work with whole blood, plasma and serum samples.
- Storage: The kit can be stored at room temperature or refrigerated (2-30°C).
- Shelf Life: 24 months from manufacture date
- Sold in packs of 100
- Forensic/Medical Professional Use Only\*\*
- Pending FDA review\*\*\*

## MORE INFO ABOUT OUR RAPID CORONAVIRUS (COVID-19) TEST KIT

For additional information, such as instructions, how-to videos, clinical trial info, the product insert, and CE documentation, ask your sales representative or visit:



\*Currently, available for pre-order

\*\*Forensic/medical professional use only: Tests should be conducted by a licensed phlebotomist, or a medical professional. Verification of use case prior to shipping is mandatory.

\*\*\*Warning: This Coronavirus instant test kit has not been reviewed by the FDA and results from antibody testing should not be used as the sole basis to diagnose or exclude SARS-CoV-2 infection or to inform infection status. The FDA has allowed the sale and use of these kits in the United States prior to FDA EUA approval, for laboratory and healthcare workers at the point-of-care, during the public health emergency. More information can be found from the FDA at <https://www.fda.gov/media/135659/download>

## AVAILABLE UPON REQUEST:

- Package Insert
- Clinical Evaluation Report
- Stability Report

“I recommend Confirm BioSciences COVID-19 tests 100%, they have a reliable product, excellent customer service, and are very trustworthy and dependable.”

— Dr. Fernandez, Third Tooth Dental Care, April 16, 2020

“We ran 12 tests on known positives (10) and negatives (2) and got 100pct concordance on the IgG with some of the IgG positives also having varying intensity of IgM. There were no isolated IgM positives but all of these people were sick over 10 days prior and now recovered.”

— Clinic in East Hampton, NY, April 16, 2020